



QRS Pen Applicator

The Magnetic Field Pen (MFP) generates a cone-shaped magnetic field from a point source, with the highest field intensity at the tip of the pen.

Advantages of Magnetic Field Pen

- Deep action (like a laser, but gentler and weaker)
- Long-lasting stimulation

Applications of MFP

Localized Application: Apply to small joints (finger, jaw, elbow, etc.) in the case of localized pain (such as tennis elbow or TMJ problems) or to the head area (eye, ear, and nasal cavity). Selection of intensity and time is the same as that for the Small Pillow. Exception: use low setting when treating areas on the head.

1. Begin with the large mat (this dramatically increases the amount of stimulation in each cell)
2. Use the pen applicator to target the specific area of the body.
3. Recommended maximum treatment time is 30 minutes.

Acupuncture: The QRS Magnetic Field Pen can help relieve stressed meridians on the body thus the pen is ideal to be used to treat acupuncture points, and children and adults who have a fear of needles. The MFP is also appropriate for acupuncture message. There is no maximum limit for treatment time.

- Mild Stimulation: preferable 45-90 seconds per acupuncture point, Program 1, Intensity setting 3-6
- Strong Stimulation and Pain Relief: preferable 45-90 seconds per acupuncture point, Program 4, Intensity setting 3-4
- Sedation: preferable 3 minutes per acupuncture point, Program 2, Intensity setting 3-6

Magnetic Field conditions on highest setting 10:

